

Sensor Care*

DO	Always use a new sensor sheath with each patient when capturing images.
DO	Grasp the USB extension cable plug (not cable) when disconnecting sensor from USB port, and pull gently. Do not unplug the sensor from the extension cable. Do not pull on the cable itself.
DO	Store the sensor with recommended wall holders when not in use. Either use the cable drops included in your order or 2 sticky backed plastic hooks spaced 2' apart at the same height off the ground to ensure no part of the sensor or cable touches the ground. **
DON'T	Coil sensor cable as this may cause kinks and irreversible damage.
DON'T	Allow cable to touch the floor where it can become tangled or stepped on.
DON'T	Pull on cable to remove it from tab, holder or sheath.
DON'T	Twist or tangle cable during use.
DON'T	Create loops or sharp bends with cable when storing sensor.
DON'T	Store the sensor in a counter or drawer where it could possibly be shut into the opening.
DON'T	Allow the sensor to hit or strike hard surfaces.
DON'T	Allow anyone to bite directly on the sensor.
DON'T	Attempt to repair the sensor or cable yourself or by a third party. This will void your warranty.

* Failure to follow these recommendations could void your warranty.

** Do not use sensor shaped wall hangers or storage units.

Questions?

Please contact technical support at (800) 704-8494 or at imaging@dentimax.com

Open Sensor Care Guide

How to care for your Open Sensor

